

# Multi-Use Barrier

(088-058) Applications



## Lamp and Operatory Toggle Switches

Place rolled cover at end of switch. Un-roll to desired length.



## X-Ray Bite Blocks (*fits Pano & other round bite blocks*)

Place rolled cover at end of block. Un-Roll to desired length.



## Ultra-Sonic Scalers, Prophy Jets, Low Speed Handpieces

Place rolled cover at sharp end of instrument. Place barrier. Un-roll cover down body of instrument.



## Contamination Free Removal:

1 - Remove glove from one hand and hold instrument. Grasp bead-lip with index finger and thumb.

2 - Pull barrier off of instrument inside-out.



## Writing Pens - Pencils

1 - Place writing tip into center of rolled cover. Un-roll up pen body. For easy removal, do not overrun top of pen.

2 - If necessary, scratch tip of pen on pad to pierce barrier end.

