

T-Bar Barrier

(088-015) - *Application*



1 - Place open mouth of cover over bottom of handle. Insert forefinger inside cover, stretch upward.



2 - Pull remainder of cover up and over handle.



3 - Pull beaded circle outward for thumb guard protection.

T-Bar Barrier


(088-015) - *Removal*



1 - Grasp beaded circle firmly between thumb and forefinger, avoid touching handle.



2 - Stretch cover up and over top of handle, insert thumb into cover

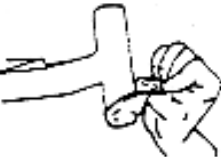

3 - To help contain gross contaminants, pull downward, turning cover inside out with thumb.